

SAFETY & STORAGE



THREAD USE & STORAGE

The foil bag contains multiple envelopes of threads. Once envelope is opened, threads must be used within 24 hours. **Once the foil bag is opened, all threads must be used in 4 to 6 weeks.** Foil bag must be airtight upon closure. If foil bag is not closed as directed, the threads will be **subject to oxidation and breakage will occur.**

PACKAGING

EuroThreads should not be used in patients with any known allergy or foreign body sensitivities to polydioxanone or poly-L-lactic sutures or in situations where internal fixation is otherwise contraindicated. The device should also, not be used in patients appearing to have very thin or soft tissue in which implant may become visible. Implantation of foreign materials into tissue can result in histological reactions. Do not use in patients that are allergic to EuroThreads.



INSTRUCTIONS

Instructions of Use: Consult instructions for use. See product Training Protocols, Medical Device Manual and other available educational material.




STERILE

The contents of each package have been sterilized by Ethylene Oxide. Product is non-toxic and non-pyrogenic meaning product is non-flammable.




STORAGE CAUTION

Use caution when storing this product. Consult accompanying documents. Do not store opened, resealed bags for more than eight weeks.

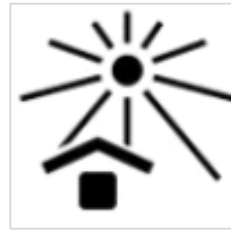

DIRECT LIGHT

Keep package out of direct sunlight. The outer re-sealable foil bag protects from direct light and humidity. Store bag in dry area out of direct sun.

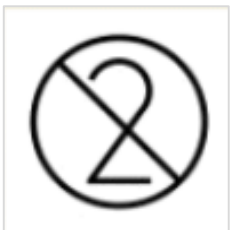

SINGLE USE

This product is for single Use Only. Please discard immediately after opening. Do not use if package is open or damaged as sterility may be compromised.



MOISTURE

Store bag in a temperature controlled room. Product is sensitive to moisture so if bag is not properly sealed, product viability will be compromised.



MANUFACTURER

The product is distributed by EuroThreads LLC, a registered Wyoming company. Please visit website provided on back label for additional information.



LOT NUMBER

Each package contains a lot number, manufacturing date and product expiration date. Information is located on bottom of the back label.



EuroThreads LLC is a medical device company known for its aesthetic enhancement technology. Providing innovative technology that enables physicians all around the world to meet their patients' needs for safe and effective aesthetics solutions. EuroThreads is committed to support medical professionals with excellent customer service and high-quality training programs.

Important EuroThreads Safety Considerations EuroThreads is a FDA approved medical device indicated for soft tissue approximation where the implantation of sutures is appropriate. Physicians should be familiar with recommended techniques involving EuroThreads' devices, as well as proper patient selection and suture placement.

EuroThreads should not be used in patients with any known allergy or foreign body sensitivities to polydioxanone or poly-L-lactic acid sutures or in situations where internal fixation is otherwise contraindicated. (e.g. infection.) The device should also, not be used in patients appearing to have very thin or soft tissue in which the implant may become visible.

After placement patients may experience a minimal acute inflammatory tissue reaction. Symptoms may include minor pain, swelling and bruising.

Material sensitivity / allergic reactions should be reported to EuroThreads LLC.


Implantation of foreign materials into tissue can result in histological reactions.


Other potential side effects include sensory nerve injury, asymmetry or banding.

Individual results may vary.


EuroThreads medical device is available only through a licensed practitioner. Complete product and safety information is available from www.EuroThreadsLLC.com or Call +1 612 800 7155 1621 Central Avenue, Cheyenne, WY 82001, USA

STERILE Sterilized by Ethylene Oxide.
Non-toxic, Non-pyrogenic

 Instructions of Use: Consult instructions for use. See product Training Manual & educational material.

 Single Use. Please discard immediately after use.

Do not use if package is open or damaged as sterility may be compromised. Keep away from sunlight.

 Storage: Caution! Consult accompanying documents.

The outer re-sealable foil bag protects sutures from light and humidity. The inner pouches are for sterility. You can store unopened pouches in re-sealable foil bag. Opened pouches should be used the same day, otherwise discard immediately. When resealing foil bag, expel all air as seal is being closed. Store bag in a temperature controlled room, away from direct sunlight.

We do not recommend storing opened, resealed foil bag and its contents for more than 8 weeks.

Distributed by: EuroThreads LLC

EuroThreads

 123-45670